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Please read the instructions carefully before using for the first time!

WARNING means: failure to follow these instructions may cause damage to the product or injury to the patient or other persons.

## 1. Specifications and Scope of Supply

Item No.:	4400400	4401100
Dimensions:	95 x 50 cm non-foldable	178 x 50 cm faltbar: 89 x 50 cm
Material Rollboard:	Special foam with wood core	Special foam
Material cover:	Sliding fabric with coated silver cover	
Colour cover:	silver	
<b>X-ray compatible</b>		
Scope supply:	Rollboard incl. cover / Manual	
	Cover:     	

**Made in Germany**

Transatlantic Rollboard Mini

Transatlantic Rollboard Classic

## 2. Use/Placement

The Transatlantic Rollboard Classic and Rollboard Mini are ideally suited for atraumatic, painless and low-friction transfer of seated and recumbent, e.g. heavy, particularly helpless or premedicated patients to a different bed, a trolley, an x-ray, examination, an operation table or a mobile shower trolley.

Transfer is also possible over a short distance or over different heights. The Transatlantic Rollboard Classic or Rollboard Mini allow a support for the transfer of patients with many drainages or with problems such as respiratory distress syndrome. For safety reasons, transfer with the Transatlantic Rollboard Classic or Rollboard Mini should be done with two persons and if applicable with a sheet for pulling the patient.

### **WARNING:**

- » All brakes have to be tightened fully!
- » Ensure the working heights of the transfer levels are correct for a working without back injury.
- » The Transatlantic Rollboard Classic and Rollboard Mini must not be pulled on the loops!

Move the patient in a recumbent position slightly to the side and slide the Transatlantic Rollboard Classic or Rollboard Mini about 1/5 of the way under the patient. Place the patient in a supine position on the Transatlantic Rollboard Classic or Rollboard Mini. Stand next to the transfer level and push the patient with the shoulder and hip on the Transatlantic Rollboard Classic or Rollboard Mini into the transfer unit. To remove the Transatlantic Rollboard Classic or Rollboard Mini place the patient briefly on the side and remove the Transatlantic Rollboard Classic or Rollboard Mini from under the patient. Always be aware of your back when working as a carer.

**This product is suitable for re-use.**

## 3. Cleaning, Care and Disinfection

The Transatlantic Rollboard Classic and Rollboard Mini can be cleaned with medical disinfectants. Do not use any abrasive or bleaching agents and sharp objects. For the use in hygienically critical areas, please use the disposable protective foil (Item-No. 4500120).

### **Option:**

The Transatlantic Rollboard Classic and Rollboard Mini can also be used with the Transa disposable sliding foil. The Transa disposable sliding foil is supplied in 100 metre rolls (item No. 4400200 or 4400203 in dispenser box with cutter). Cut off the appropriate length of Transa sliding foil from the roll. Pull the sliding foil over the Transatlantic Rollboard Classic or Rollboard Mini. With its outstanding sliding properties the Transa disposable sliding foil used with the Transatlantic Rollboard Classic or Rollboard Mini makes the most difficult transfers easy.

### **IMPORTANT NOTE:**

- » When using the Transatlantic Rollboard Classic or Rollboard Mini with the Transa disposable sliding foil (the sliding foil is disposable and not washable) please note the special instructions of the Transa disposable sliding foil.

## 4. Warranty

The warranty is valid for a period of 3 years. It does not cover any defects which are due to improper use. Transatlantic cannot be liable for any accidental or indirect injury or damage.

## 5. Visual and Function Inspection/ Safety Information

Repositioning products are also subject to natural aging. Check the equipment regularly and carefully to make sure it is working properly. Make sure that the product is in good condition and that the material and seams are not damaged.

### **WARNING:**

- » Defect products must no longer be used.
- » Make sure that there are sufficient number of carers and assistants corresponding to the transfer situation.
- » Never leave the patient unattended.
- » The smooth surface considerably minimises resistance to friction during the change of position. Carefully check the two transfer surfaces, making sure they are stable and brakes are applied (bed, stretcher).
- » The distance between free transfer surface using the Transatlantic Rollboard Classic or Rollboard Mini must not be more than 15 cm.
- » Make sure that carers are always aware of their backs during the transfer.
- » Do not use sharp-edged items when using the Transatlantic Rollboard Classic or Rollboard Mini. The both Rollboards could get damaged in this case.
- » Transfer under uncovered skin must be done with a cloth between the Transatlantic Rollboard Classic or Rollboard Mini and the patient (e.g. towel).

## 6. Product Labeling

The Transatlantic Rollboard Classic and Rollboard Mini are medical products, class I and they comply with MDD directive 93/42/EEC. Transatlantic is certified in accordance with directive DIN EN ISO 13485 and DIN EN ISO 9001.

## 7. Disposal Information

---

For the Transatlantic Rollboard Classic and Rollboard Mini there are no special disposal rules, they are disposed of in accordance with national and local regulations as necessary.

## 8. Accessories

---

Transa - Slidefoil

**Item-No. 4400200**

Optionally available in dispenser box with knife

**Item-No. 4400203**

Disposable protective foil

**Item-No. 4500120**

If you have any questions about the use of our products, please call our service number:

**+49 (0)6081 9430-50**

Transatlantic Handelsgesellschaft Stolpe & Co. mbH  
Siemensstraße 21-23  
D-61267 Neu-Anspach  
info@transat.de  
www.transatlantic.de

---

"Information subject to technical modifications to our products and deviations in colour or printing errors"

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