
REF	Désignation / Omschrijving Bezeichnung / Designation	Mesures / Maten / Maßnahmen / Measurements
RFW16905	PEDIATRIC WRIST HOLDERS	0-2 ans/jaar/jahre/years
RFW16910	PEDIATRIC WRIST HOLDERS	2-5 ans/jaar/jahre/years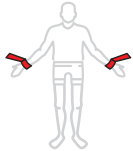

Lire la notice (IFU) / Lezen IFU / Lesen IFU / Read IFU

Regarder la Video / Bekijk Video / Shau Video / Watch Video

Utilisation au lit / gebruik in bed / im Bett verwenden/ using in bed

Ceinture poignet (paire) / Polsriem (paar) / Handhalterung (paar) / Wrist holder (pair)

Lit en position basse, barrière de lit en position haute, les patients doivent être observés d'une manière constante.  
 Bed in lage positie, bedhek in hoge positie, patiënten moeten constant worden geobserveerd.  
 Bett in niedriger Position, Bettgitter in hoher Position, Patienten sollten ständig beobachtet werden.  
 Bed in low position, bed rail in high position, patients should be observed constantly.

Prescription Médicale obligatoire / Doktersvoorschrift nodig / Ärztliche Verschreibung / Doctor's prescription needed

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### Indications

1. RENOL products are used for a medical purpose to ensure the safety and security of the patient in bed and in stretcher.
2. RENOL products are only used under medical prescription.
3. The products must be used by trained staff following the RENOL's instructions for safe use and according to the establishment's safety rules.

### Precautions of use

4. To avoid any accident and in order to ensure the person's comfort and safety, it is essential to read the instruction leaflets carefully and understand them before operating Renol products. All users follow the instructions and procedures, patients' safety depends on using belts and the locking system correctly.
5. Ensure that a RENOL product is not damaged before use, all products must be clean and in good condition.
6. We recommend monitoring the immobilised patient frequently; restless and combative patients must be under constant observation.
7. Ensure that RENOL products do not come into contact with injured skin, do not obstruct breathing nor impede blood pressure.
8. Always use the correct size: too small or too large may compromise safety or comfort. Measure and select the appropriate RENOL product.
9. Remove all sharp objects from the patient (jewellery, watches, glasses, etc.).
10. Only use products RENOL on suitable support.
11. Connect RENOL products to a solid part of the chair. The straps must be secured; they cannot slide or change position; the fastening point of the strap on the frame equipment must not damage the product (sharp edge, etc.).

### Maintenance and materials

14. Follow the washing instructions marked on the products.
  15. Close Velcro fastenings before washing and drying to prevent them wearing.
  16. Detergents: avoid using chlorine.
  17. Temperature: maximum temperature 95°C.
  18. Drying: delicate cycle at low temperature, about 5% shrinkage possible.
  19. Material: 100% Terylene (polyester).
  20. The raw materials are treated with fire retardant and anti bacterial.
- All our products are CE-approved - Class I medical devices.  
For more information, contact your support specialist, your distributor or [www.renolcare.com](http://www.renolcare.com)